AXP

AxioPrisa

+

AlliedStar

Manual

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

This screenshot shows the same AS CONNECT interface as above, but with a 'Neuer Fall' dialog box open in the center. The dialog box has two input fields: 'Patientenname *' and 'Fallname *'. At the bottom of the dialog, there are 'Abbrechen' and 'OK' buttons. The background table of cases is dimmed.

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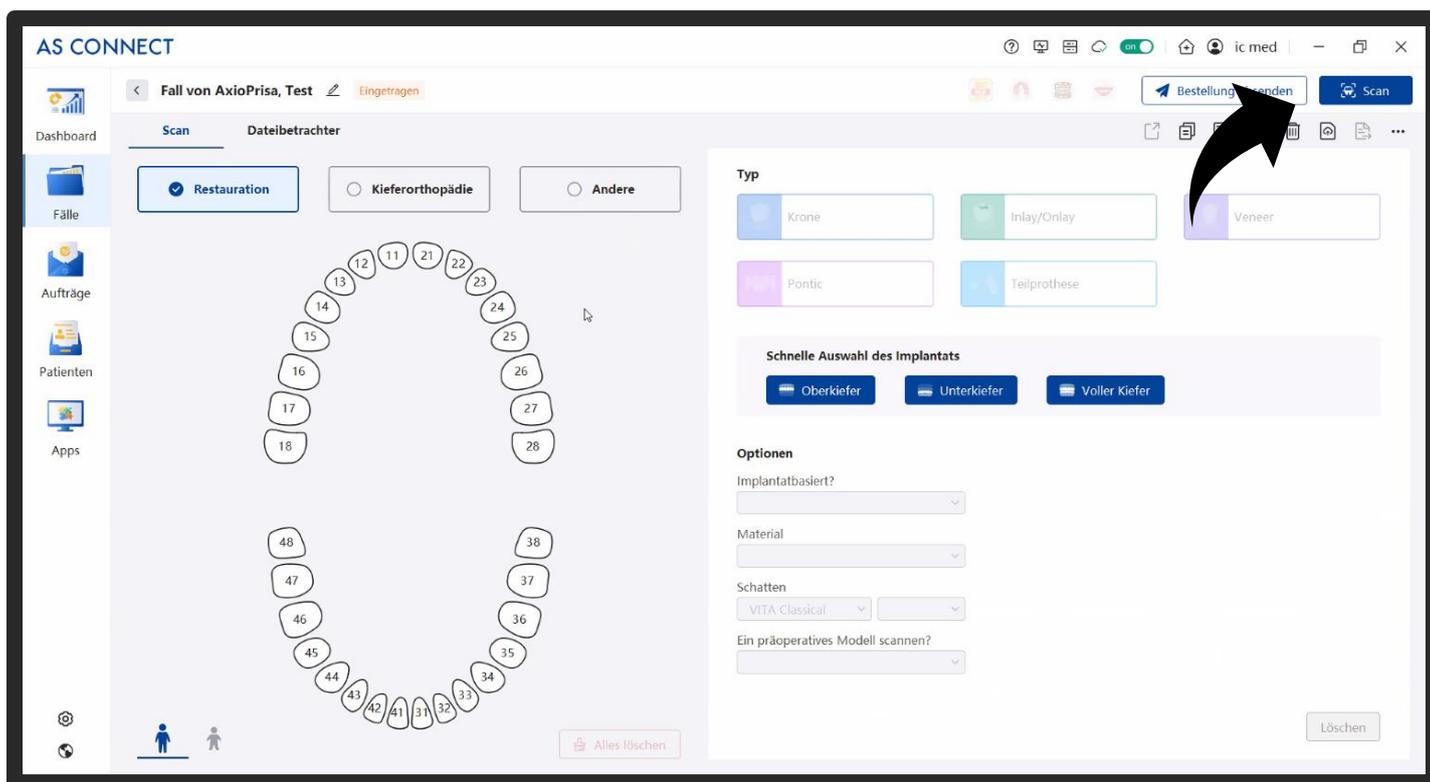
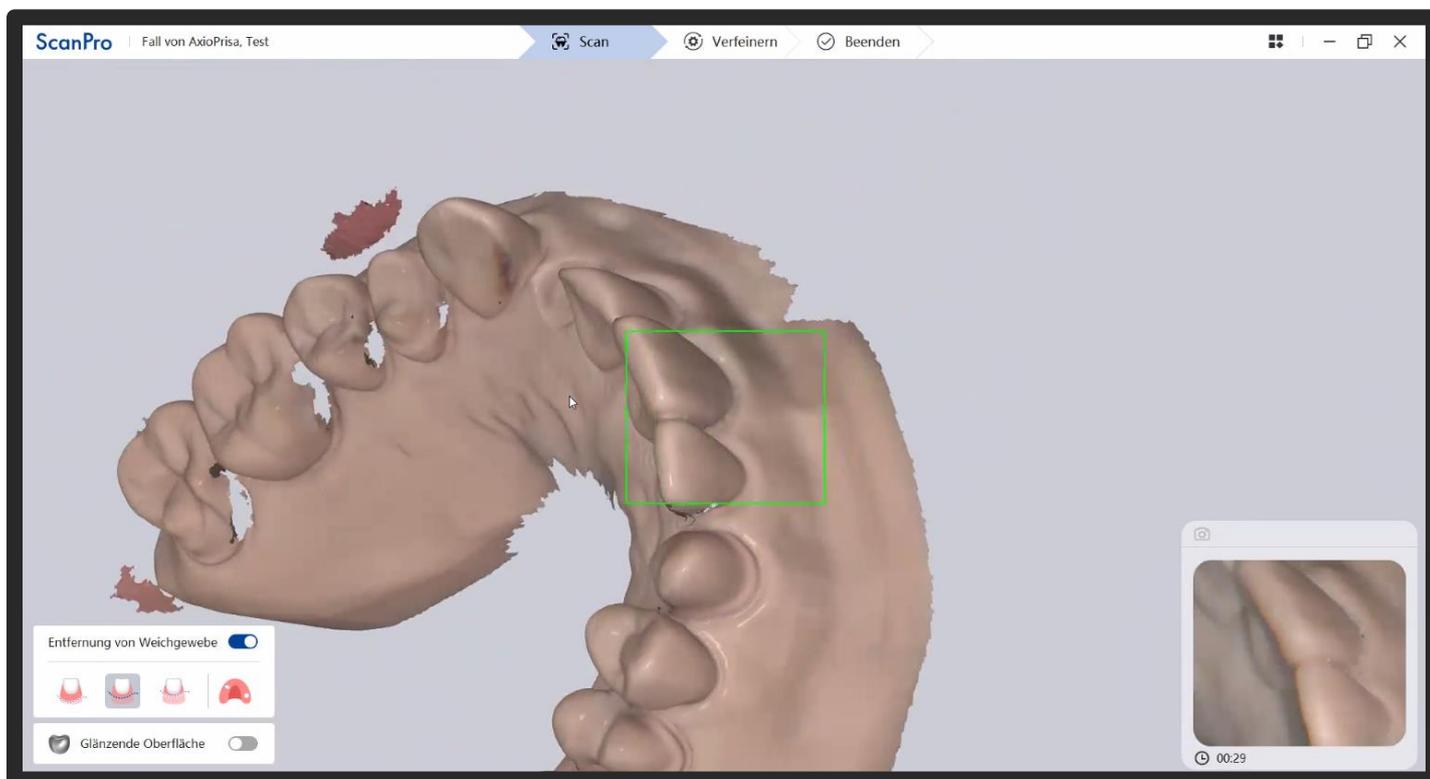
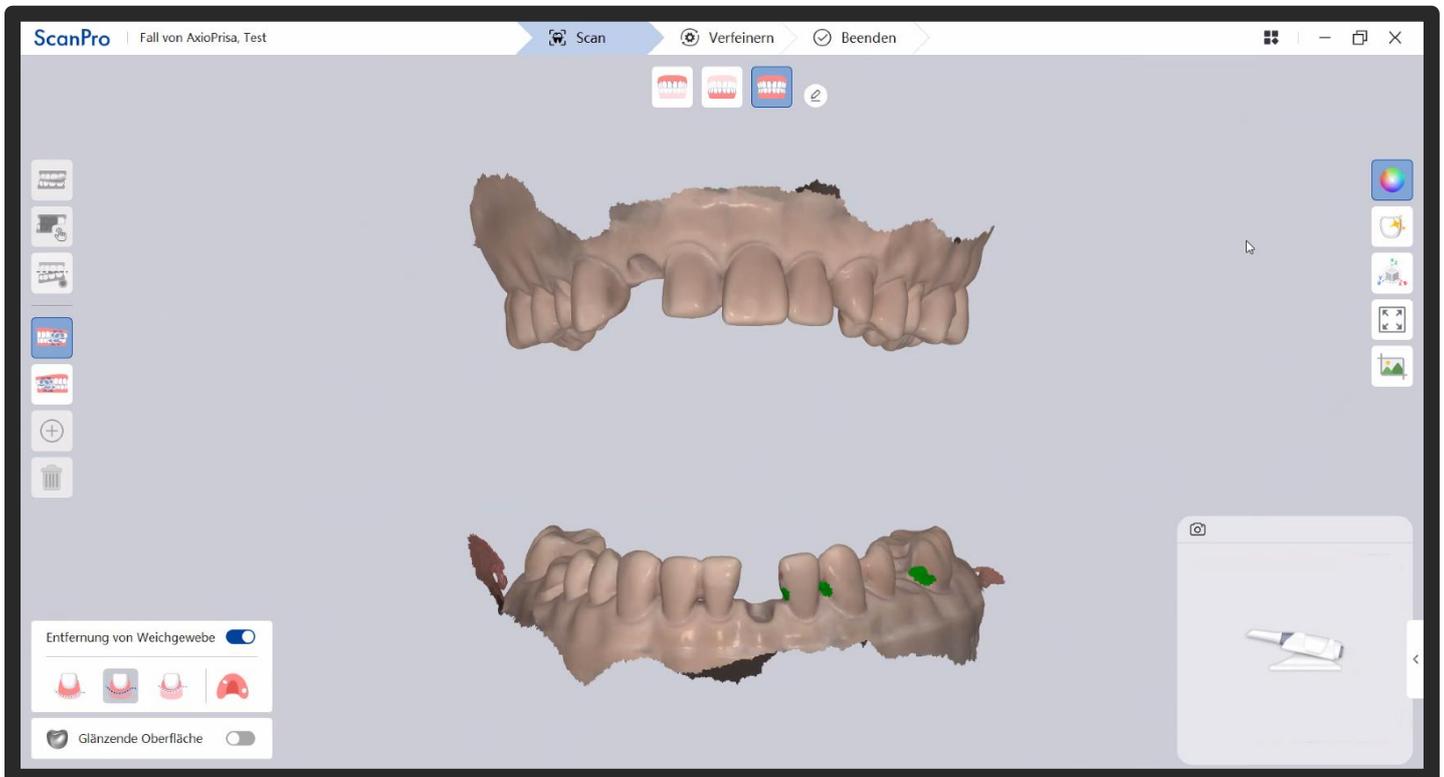
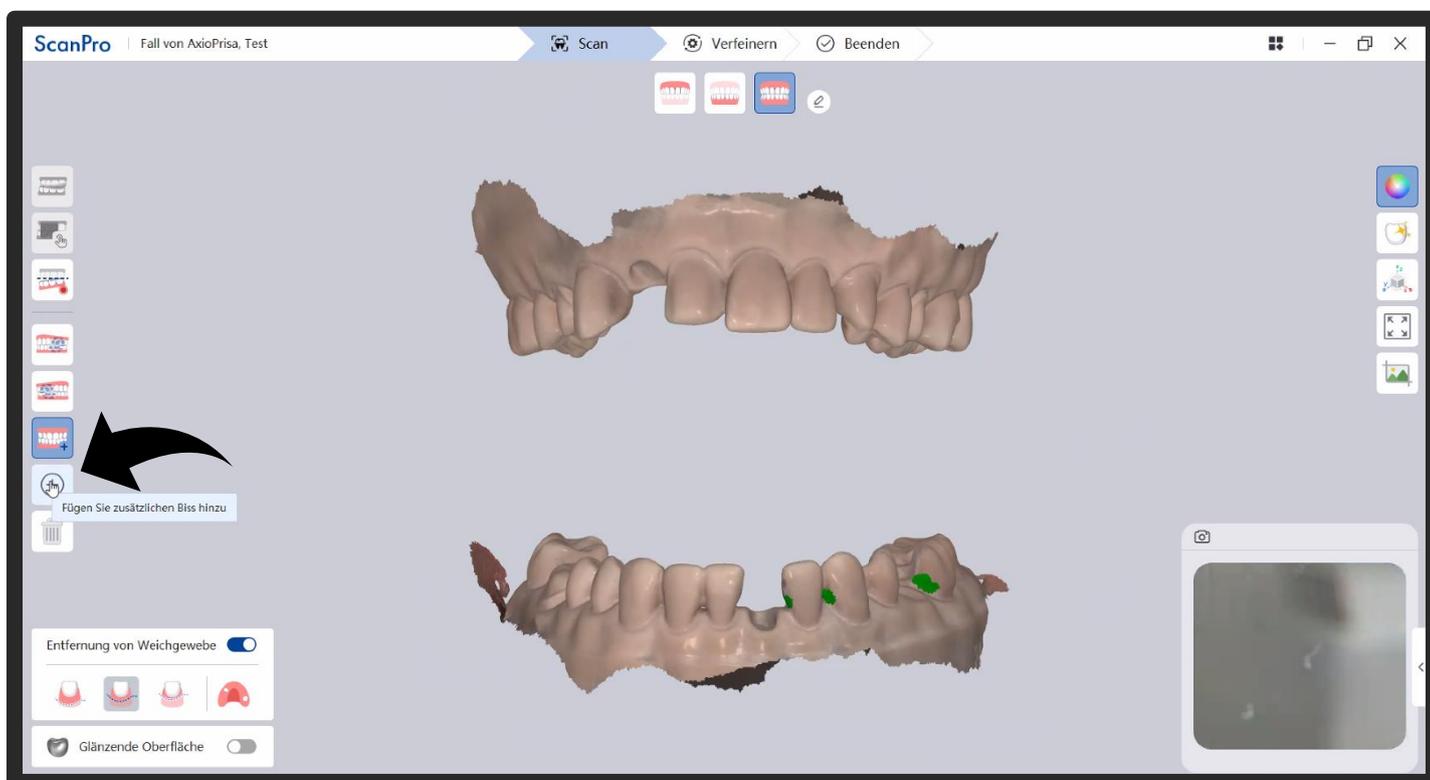
*Now the AxioPrisa transfer fork assembly can be scanned
It is important to record from the occlusion to the landing pad on the sagittal rod!*

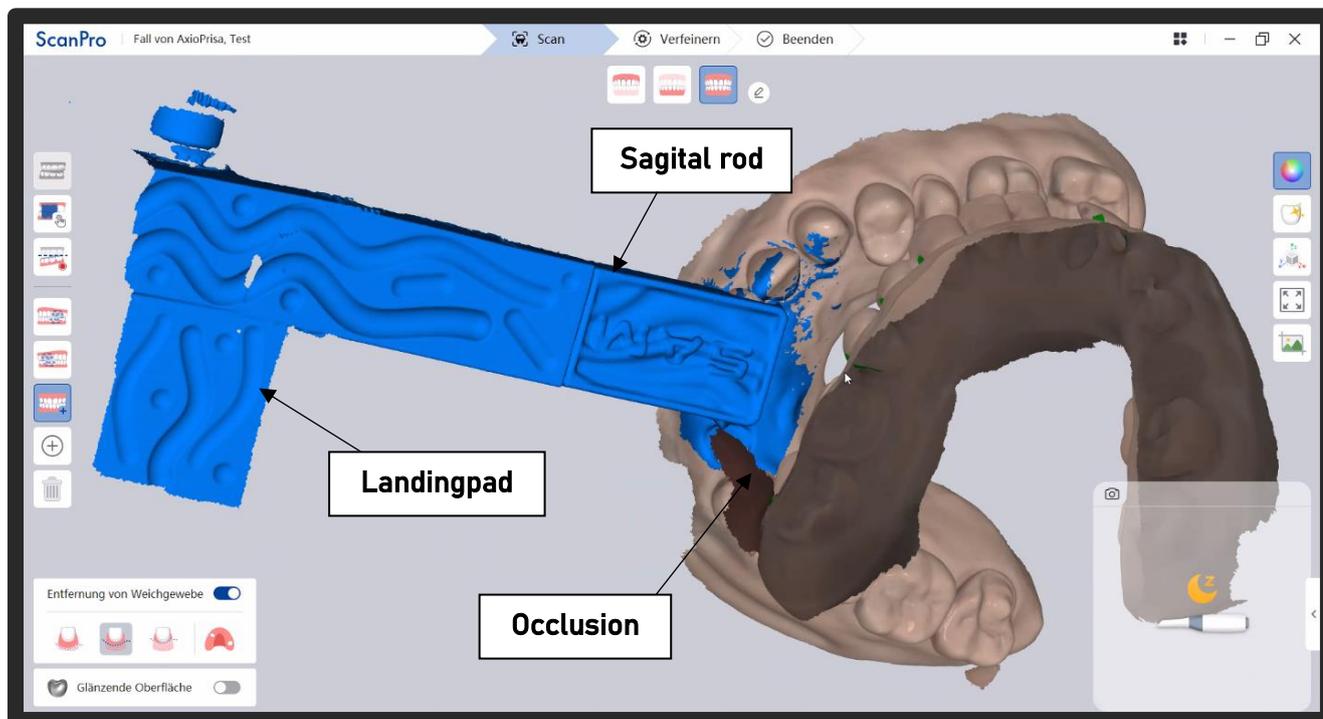After adding the second bite registration, it is recommended to start the scan in the anterior region.

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
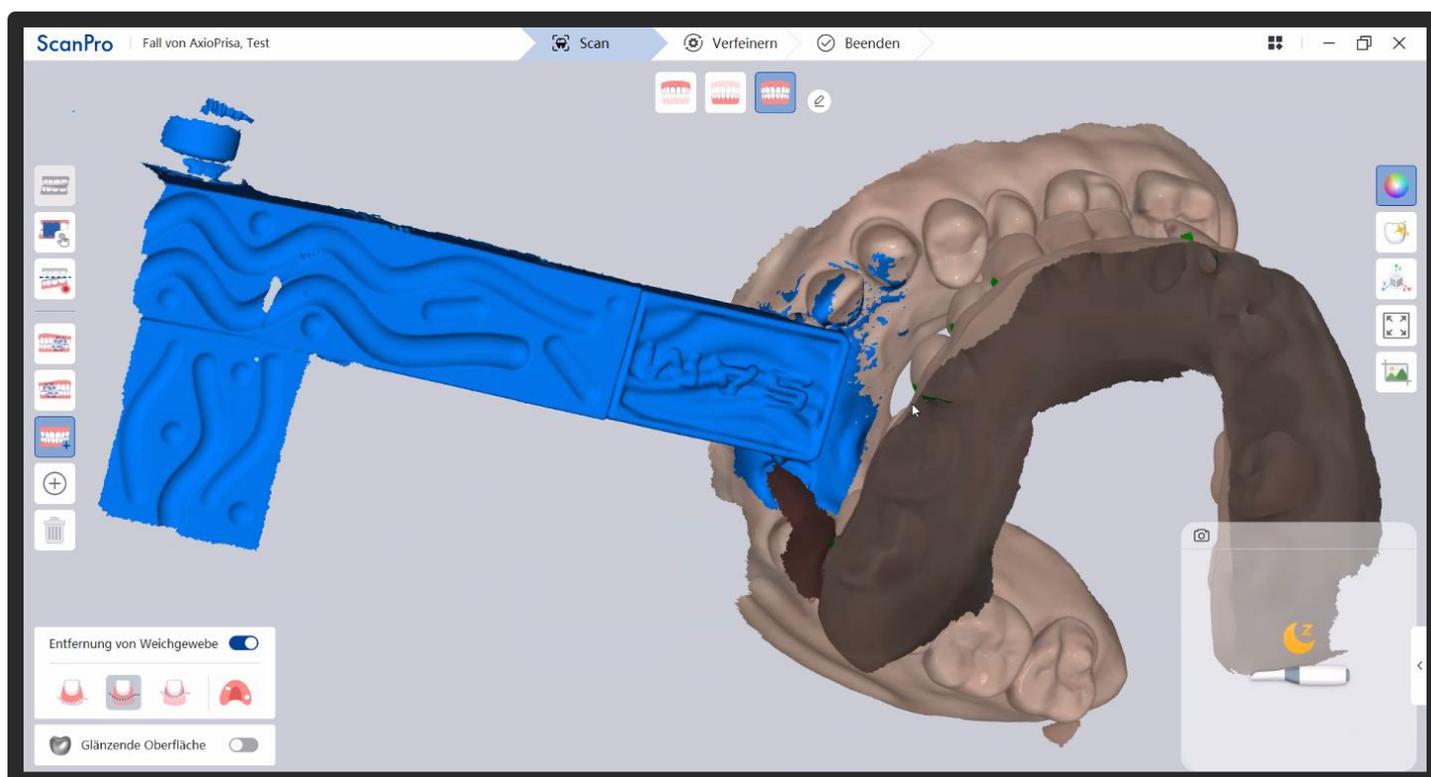
Now place the transfer bow with the AxioPrisa transfer fork assembly on the patient.

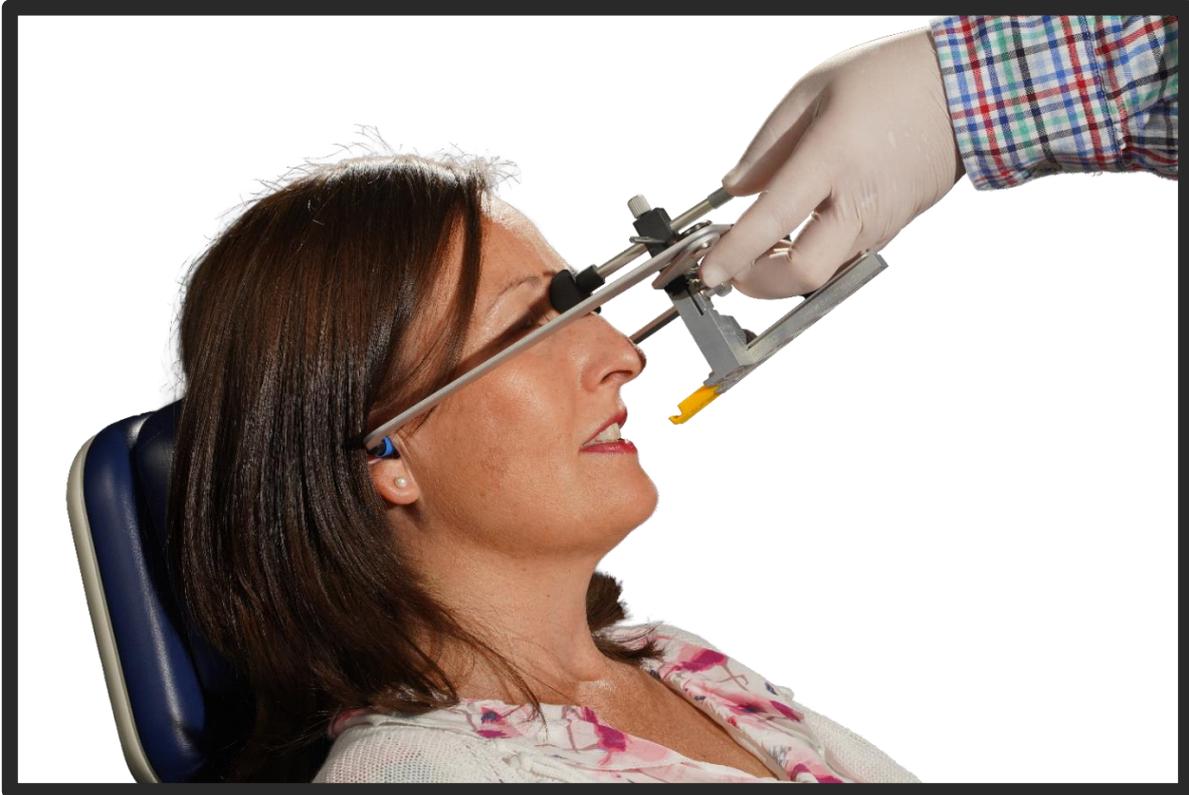Instructions for using the AxioQuick transfer bow can be found here: <https://www.sam-dental.de/downloads/>

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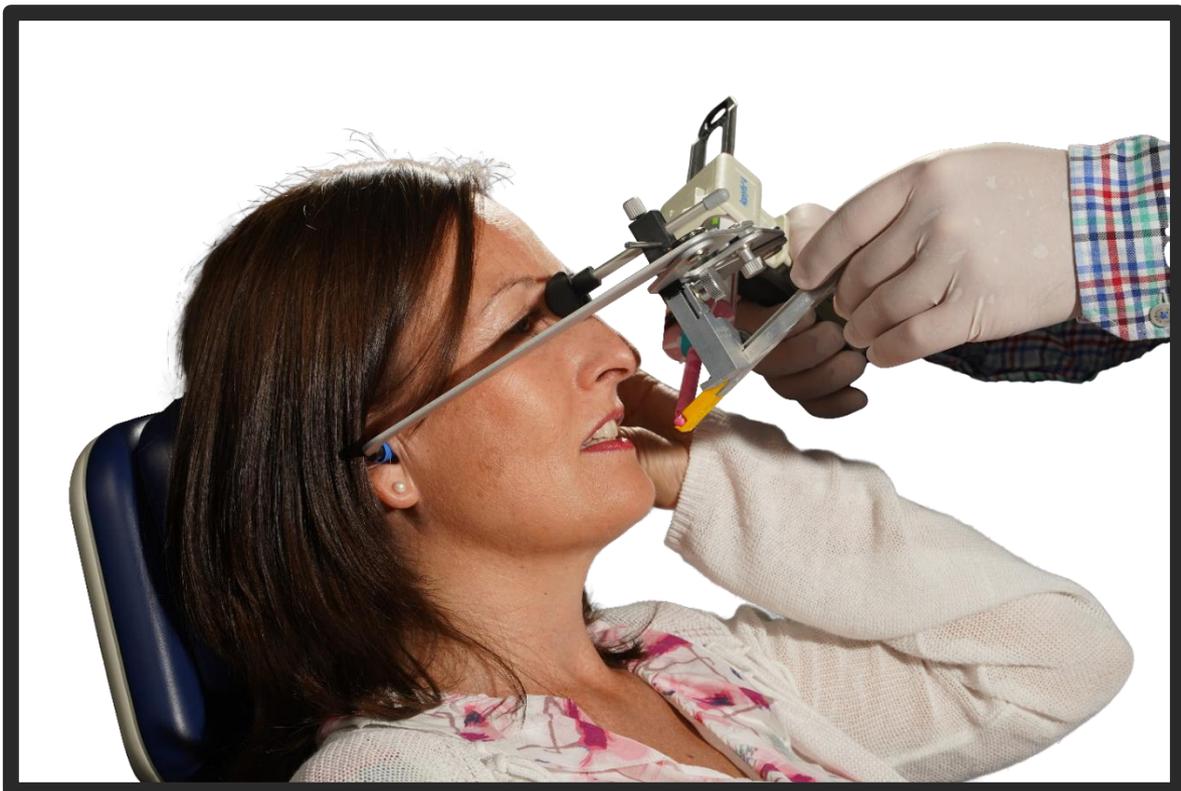
Put a small amount of silicone into the recess of the AxioJig.

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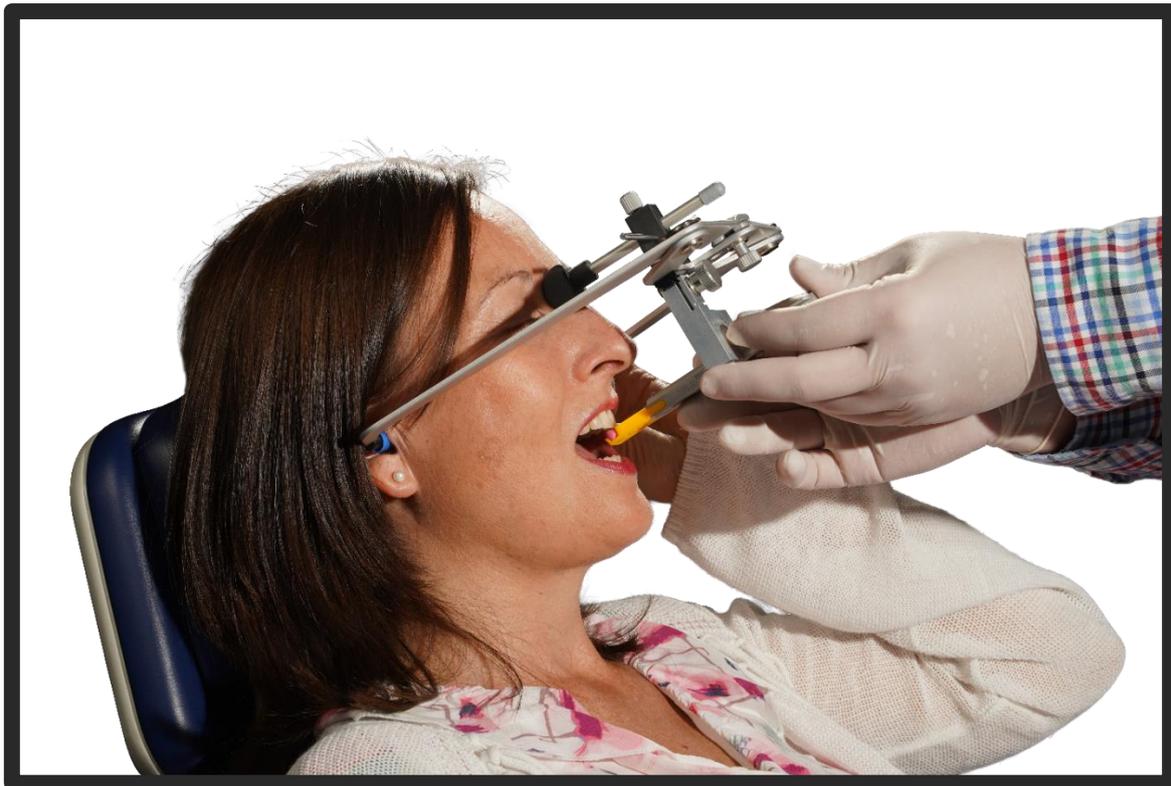
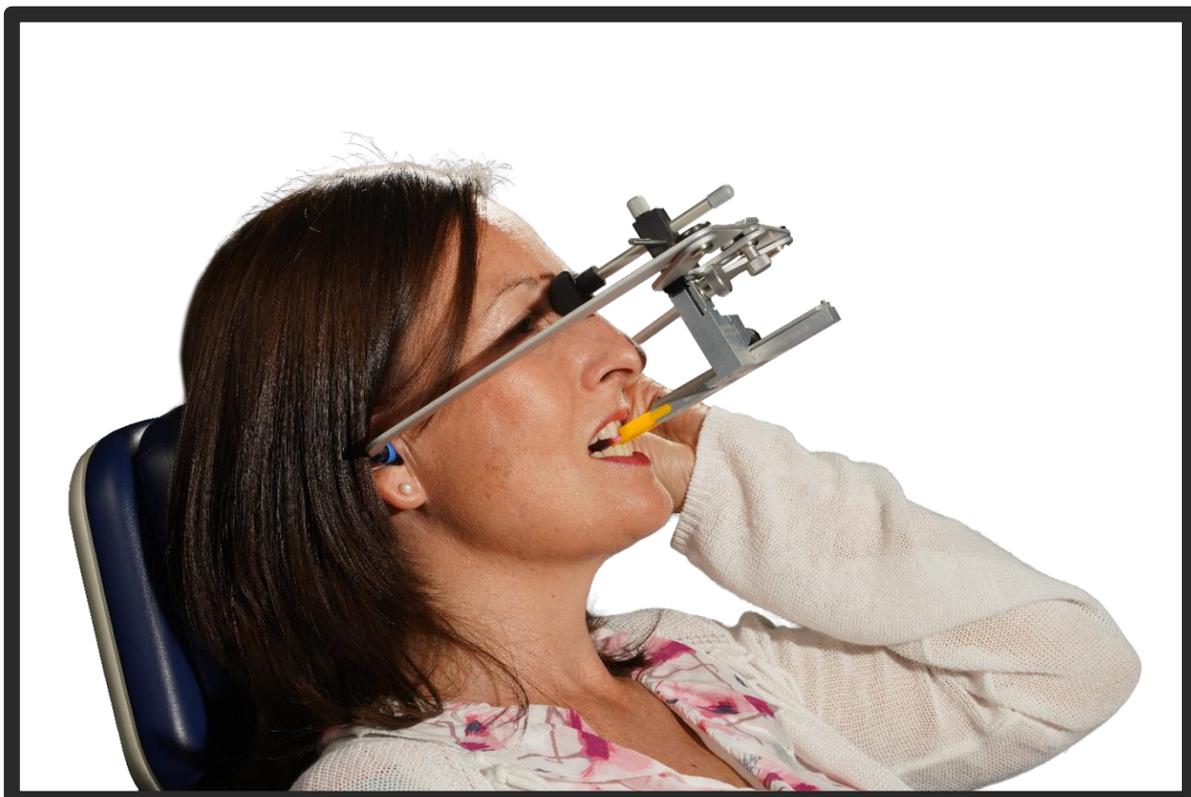
The patient must now bite into the silicone and fix the AxioJig with the lower jaw. When the silicone is hard, close the screw on the right side of the sagittal bar and the patient can open the mouth. Now the AxioPrisa can be scanned.

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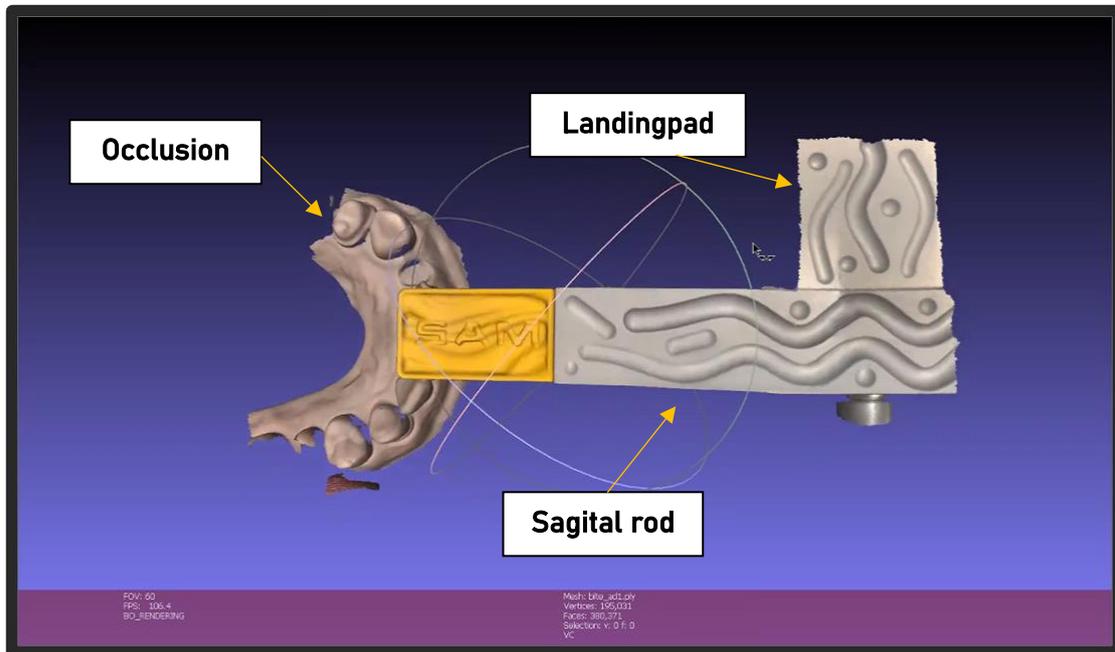
After scanning, there must be a connection between the occlusion and the loading pad at the end of the sagittal rod.

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
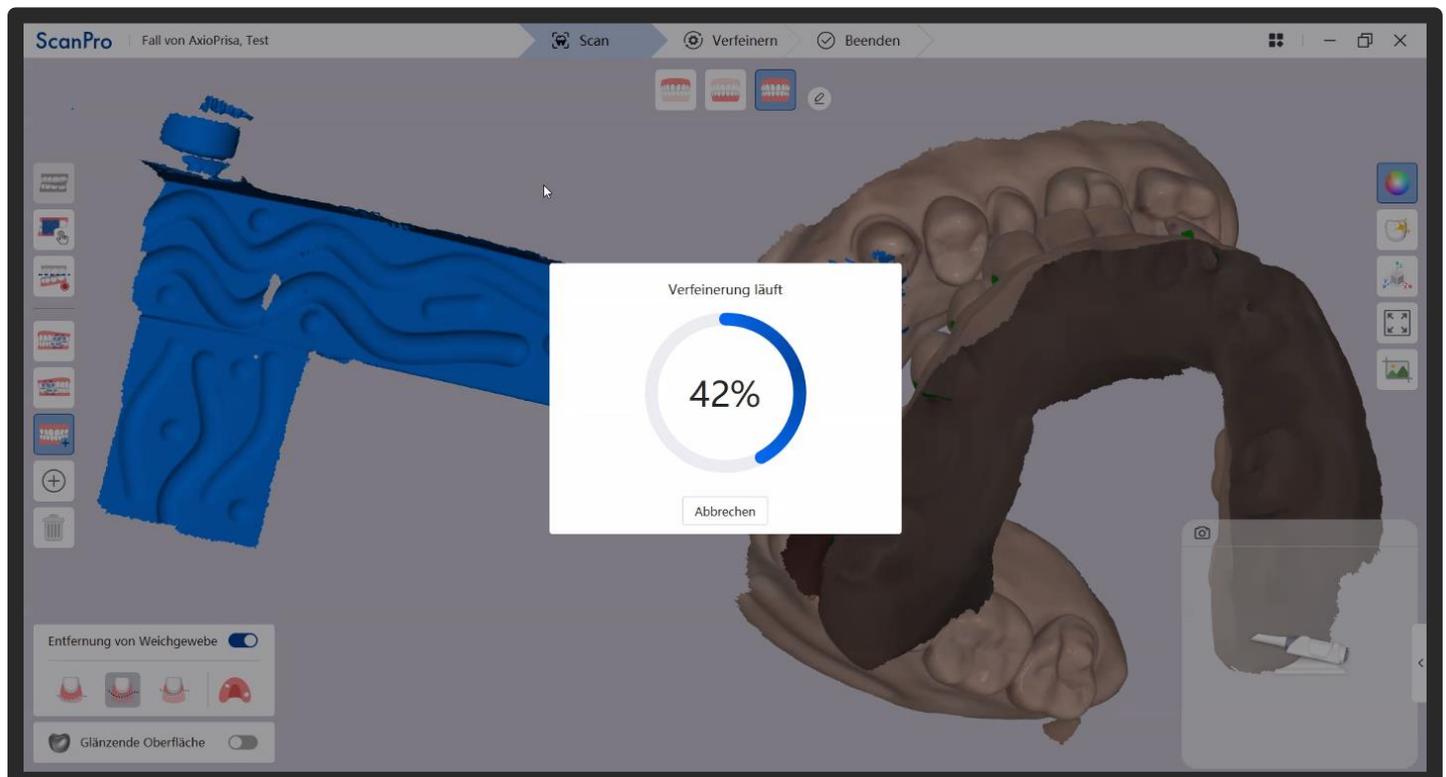
After finishing this process you need to end the case.

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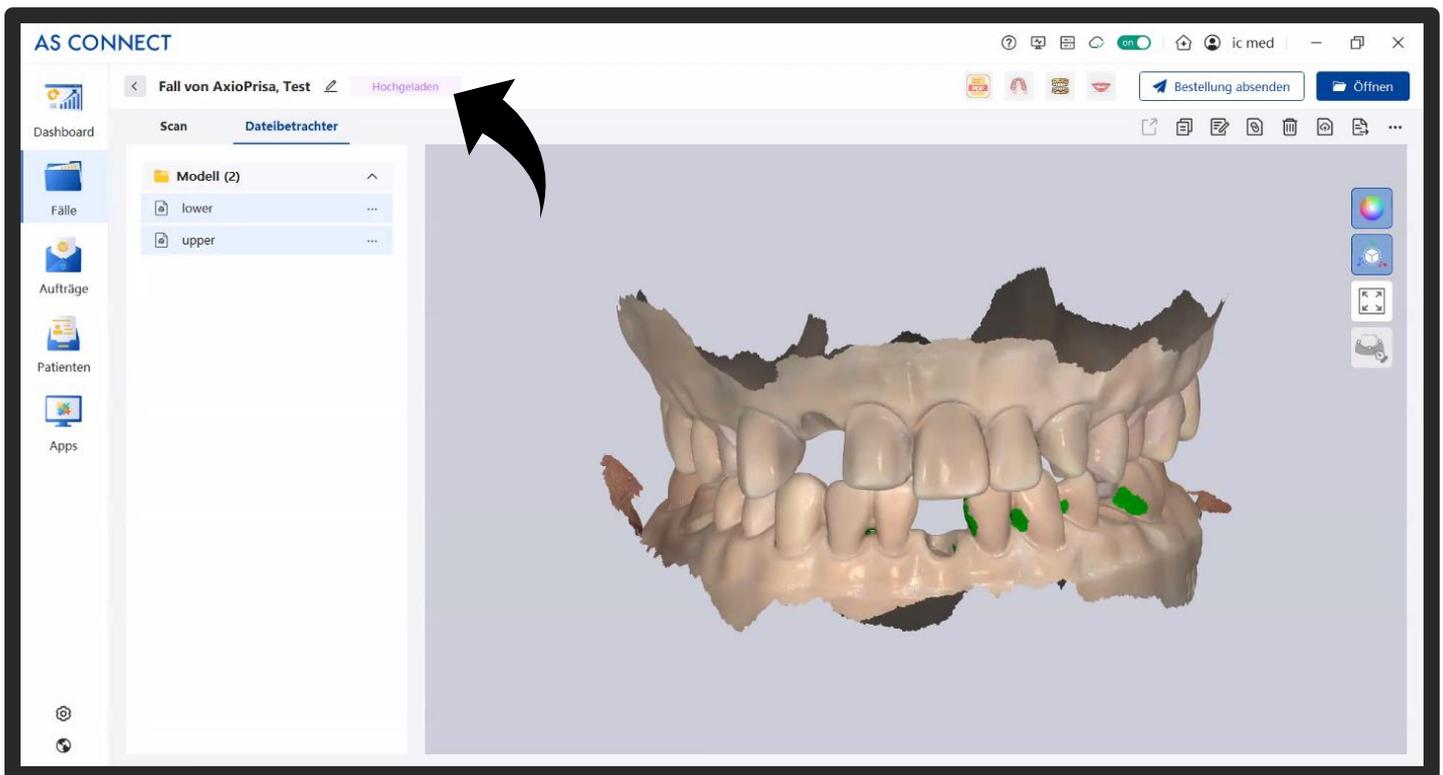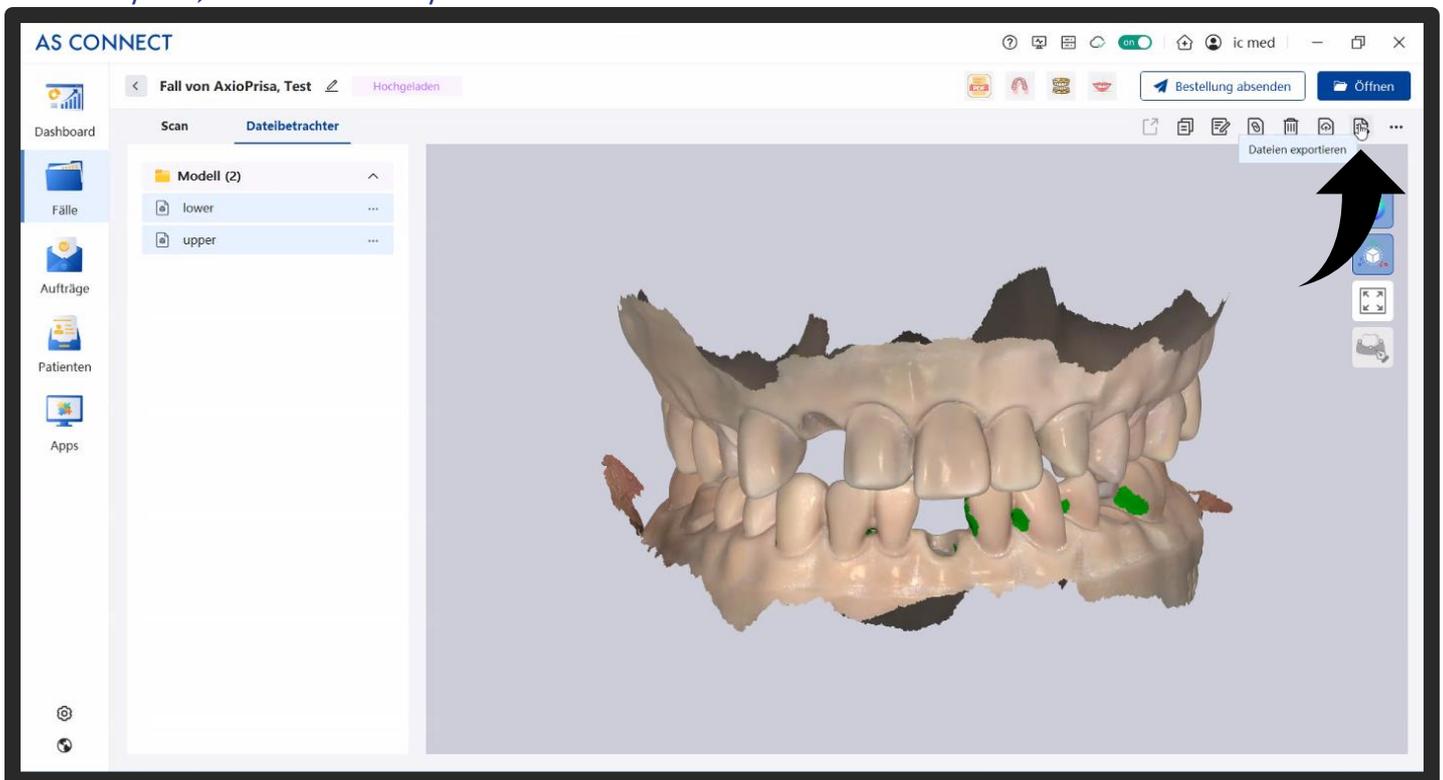
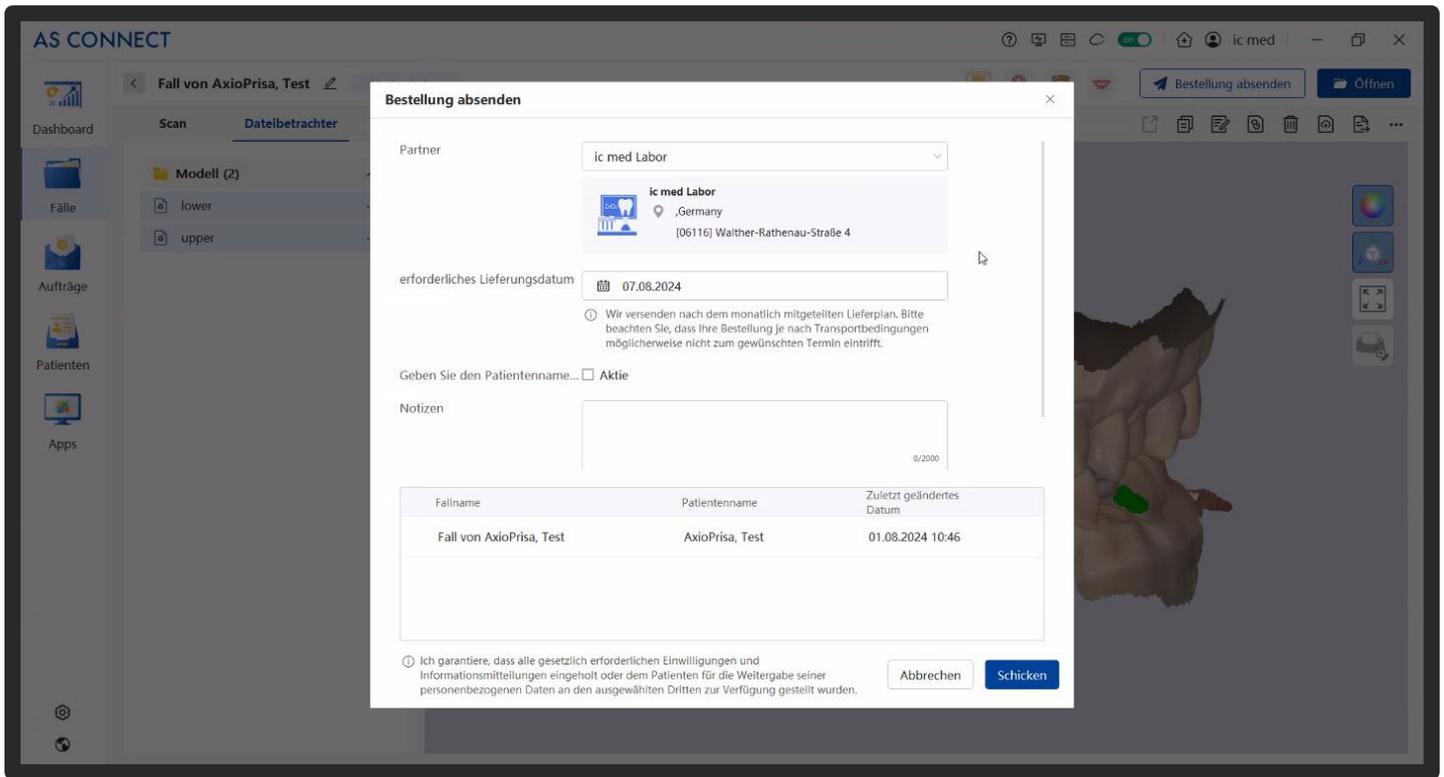
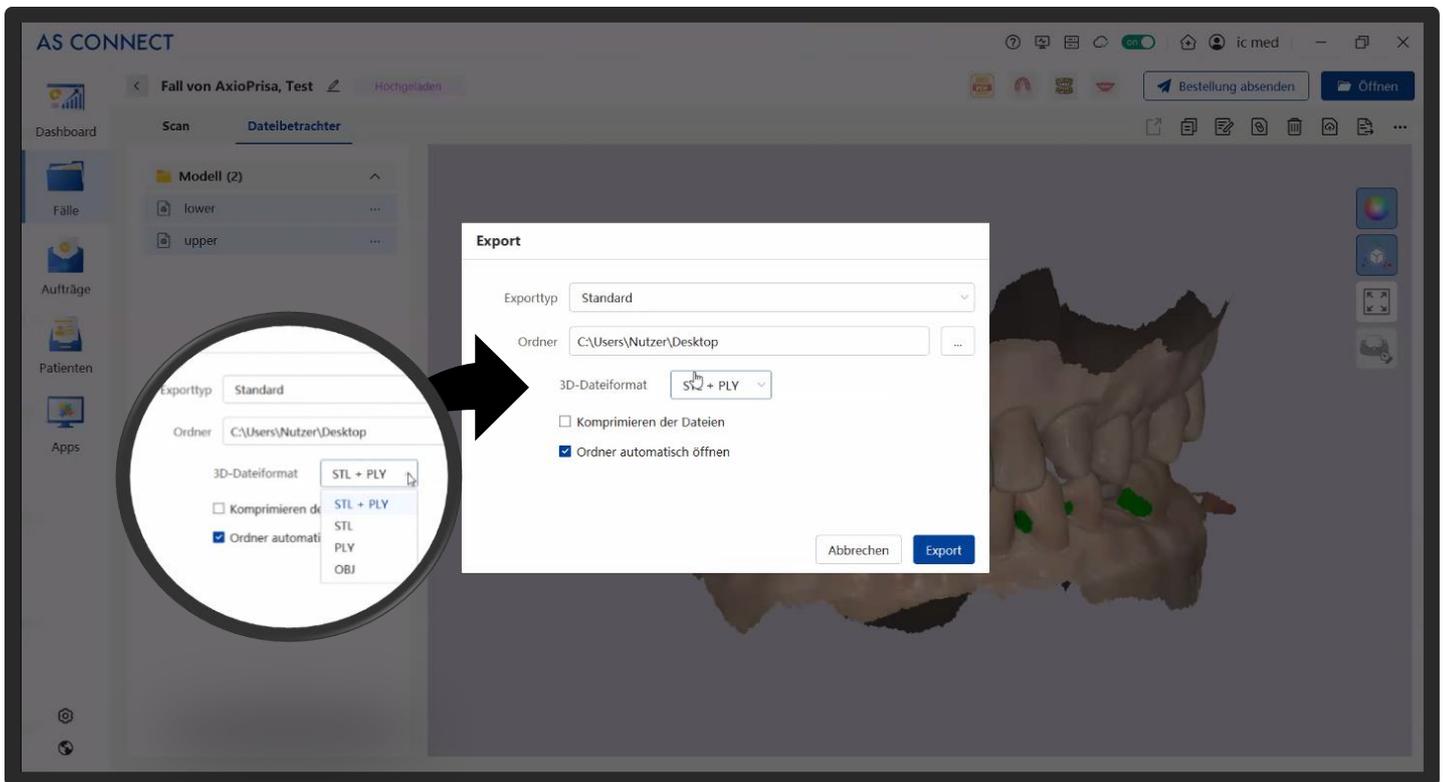
Please select the file format and storage location for the export. Select the correct file type for your printer or CAD software.

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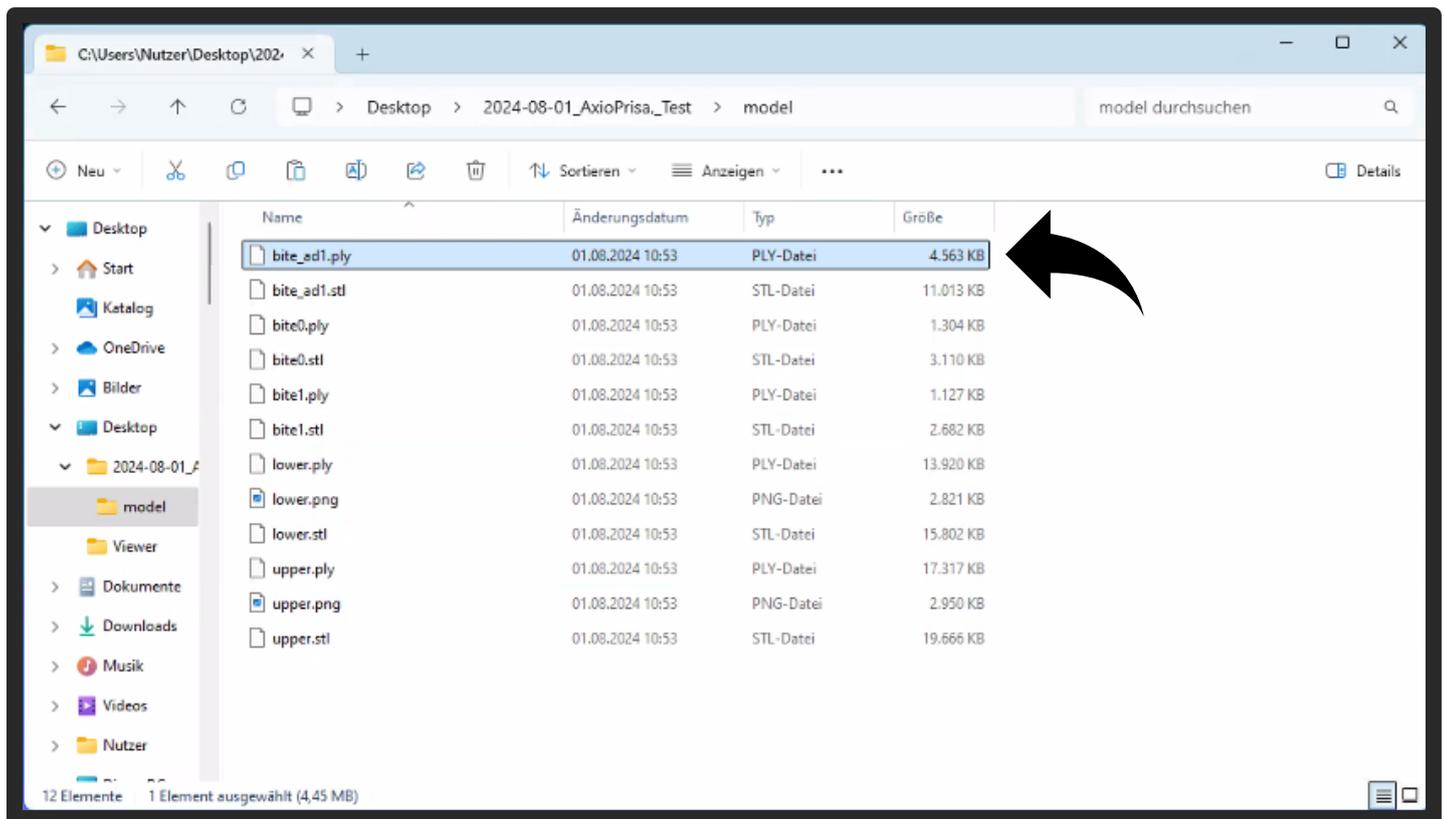The AxioPrisa scan is saved as "bite_ad1.".*

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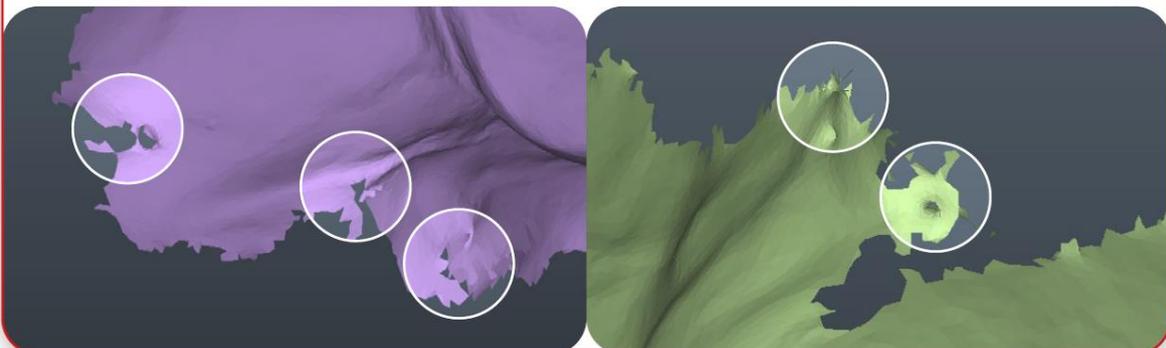
! Important note on processing scans in AxioSnapMount !

Due to possible artifacts or errors in the scan, the generation process may show an error. To prevent this, it is important that the scan of the maxilla as well as the mandible is therefore freed from errors (cleaning the scans in the scan software / CAD software).

Holes in the scan or superimposed surfaces are susceptible.

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All enquiries can be made conveniently
via the online Help Centre:

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For general questions about the product:

info@smart-dental.de

In case of technical problems:

it-support@sam-dental.de